

**READ INSTRUCTIONS BEFORE USE**

**TransLite LLC**  
345 Commerce Green Blvd  
Sugar Land TX 77478 USA  
+1 281 240 3111  
www.veinlite.com  
info@veinlite.com

**Caution**

Veinlite NEO is designed for external examination only.

To avoid contamination of the device, Veinlite NEO must be used with the disposable plastic cover.

To avoid the spread of infection, always change the disposable plastic cover between patients and wipe the new cover with a recommended germicidal wipe.

Always turn the LEDs on after the device is in position. Do not look directly at the LEDs when they are illuminated or shine the light from them directly into anyone's eyes.

Do not immerse the device in liquid.

Do not autoclave the device.

**Introduction**

The Veinlite NEO is a hand-held battery powered transillumination device for visualizing veins and arteries in neonates.

**Applications**

To assist in venous and arterial access in neonatal pediatric patients.

**Operating Directions**

**Device:**


- 1 Place Veinlite NEO against the skin with the LED Window [A] in contact with the skin.
- 2 Depress the End Button [B] to turn on the LED.
- 3 Apply gentle pressure to the Veinlite NEO to maintain optimal skin contact.
- 4 Device will turn off after cycling through the colors or you can depress and hold the End Button [B] for one second to turn device off.
- 5 When you turn on a device with a low battery the device will shut itself off.
- 6 Device will shut off automatically after 2 minutes.



**Disposable Covers:**

The Veinlite NEO should always be used with a single-use disposable plastic cover to protect it from contamination and to prevent the spread of infection between patients.


- 1 Place the Veinlite NEO inside the disposable plastic cover. Check the cover to make sure it is intact and the flap is on the side away from the patient.
- 2 These covers are not sterile. Disinfect the cover with 70% isopropyl alcohol or other medically accepted disinfectant.
- 3 Discard the single-use disposable plastic cover after each patient.

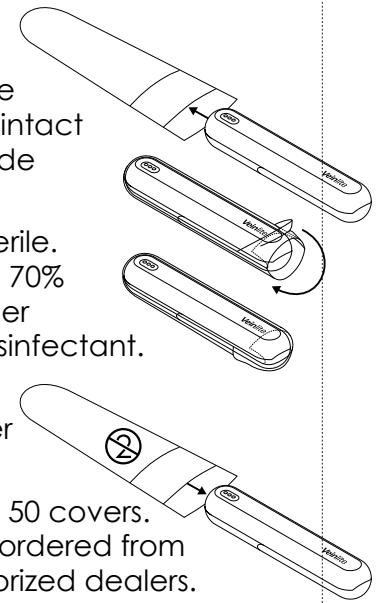

The device is shipped with 50 covers. Additional covers can be ordered from TransLite or any of its authorized dealers.

**Brightness Control:**

Depress the Side Button [C] to change the brightness. Each depression of the button will cycle through low, medium, and high brightness levels. The lowest brightness level is the default setting.

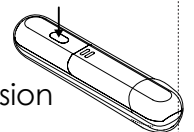

## Selecting Colors:


Depress the End Button [B] to cycle through these modes:

**Orange** Optimal wavelength for highlighting the contrast between the venous blood vessels and surrounding tissue.

**Green** Assist in finding arteries and veins.

**White** Exam light.

**Off**

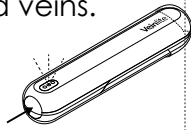

## Cleaning Directions

Cleaning the device will not be necessary if properly used with the disposable plastic cover.


In the event that the device needs to be cleaned, use germicidal wipes sparingly.

Do not submerge the unit in liquid.

## Protective Cap:

The purpose of the Protective Cap is to protect the End Button [B], Side Button [C], and LED Window [A].

Keep a disposable plastic cover on device to ensure a proper fit.

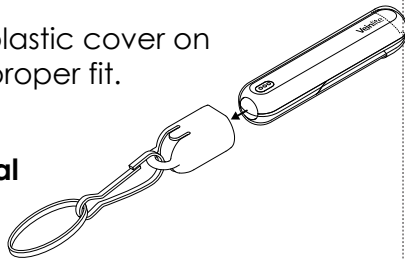

## Attaching an optional Lanyard:

Attach lanyard to the Cap [E] then snap it over the End Button [B].

The Protective Cap and lanyard should be cleaned regularly with alcohol or germicidal solution.

## Installing/Replacing Batteries:



The Veinlite NEO is powered by a single AAA battery.

1 Press firmly on the Battery Cover [D] near the seam and slide it off.

2 Remove the used battery.

3 Install a new AAA battery, oriented as indicated by the markings inside the battery compartment.

4 Slide the Battery Cover into place.

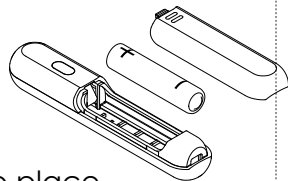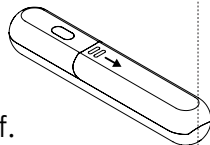

## Warranty

The Veinlite NEO has a one year parts & labor warranty.

## Troubleshooting

Before returning the device to TransLite for repair, please call or e-mail to obtain an authorization number and shipping details.

**No refunds will be given for devices shipped without authorization.**

## LED Failure

If one of the LED fails to turn on please contact TransLite for assistance. Please do not attempt to replace any LEDs.

## No Light From Any LEDs

Replace the battery, then try turning the unit on again. If the device still will not turn on, please contact TransLite for assistance.

## Specifications

Number of LEDs:

**1 Orange**

**1 Green**

**1 White**

**29g**

Weight with battery:

Dimensions:

**96.1×19.0×16.5mm**

Battery:

**AAA 1.5V**


## Explanation of Symbols

 Model Number

 Serial Number

 Manufacturer

 EC Representative (EU Only)

 Do not dispose of as unsorted municipal waste



## M. Devices Group

Health Education Centre  
The Church, Portland Street  
Southport PR8 1HU  
United Kingdom  
+44 1704 544 944

Veinlite NEO® is a Registered Trademark.

Made in USA 

\* Patent Pending

VLN DFL 0818